



# NIGHT ✓ CHECK Instructions (P/N PM1050)

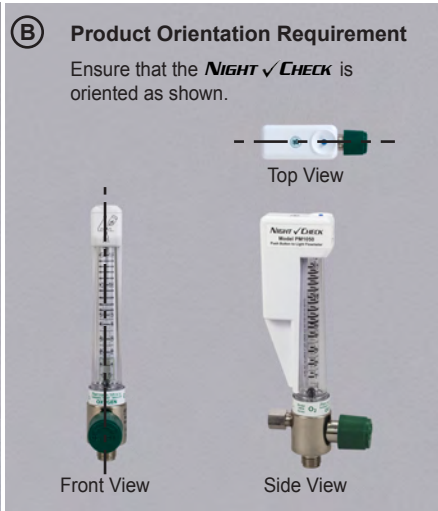

**Note:** Before removing blue protective cover from **Adhesive Pad**, practice attaching the **NIGHT ✓ CHECK** to flowmeter as detailed in Steps 1 and 2.

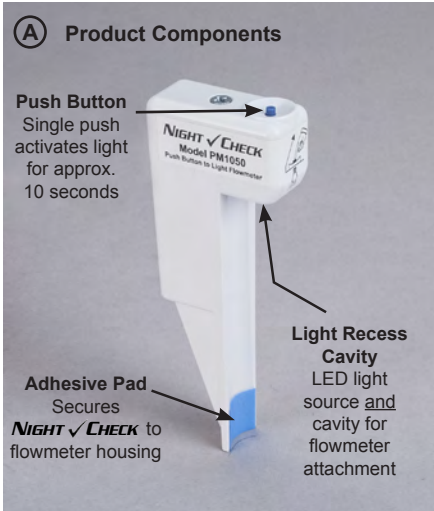

## CLEANING INSTRUCTIONS

Clean the device externally with a mild detergent or disinfectant cleaner.

## WARRANTY

Precision Medical warrants that the **NIGHT ✓ CHECK** is free of defects in workmanship and/or materials for a period of one (1) year.

**Note:** **NIGHT ✓ CHECK** is designed to fit on Precision Medical Chrome Body Flowmeters only.

## CAUTION

- DO NOT AUTOCLAVE
- DO NOT GAS STERILIZE WITH ETO (ETHYLENE OXIDE).
- DO NOT CLEAN WITH AROMATIC HYDROCARBONS OR ALCOHOL BASED PRODUCTS.
- DO NOT IMMERSE FLOWMETER IN ANY KIND OF LIQUID.