

# USER MANUAL

## DESKTOP CHARGER

Model: 508306



Batteries sold separately

### SAVE THESE INSTRUCTIONS


Read this manual before operating the device,  
save this manual for future reference.

**PRECISION**  **MEDICAL**®

300 Held Drive  
Northampton, PA 18067 USA  
**ISO 13485 Certified**

Tel: (+001) 610-262-6090  
Fax: (+001) 610-262-6080  
[www.precisionmedical.com](http://www.precisionmedical.com)

# SET-UP GUIDE

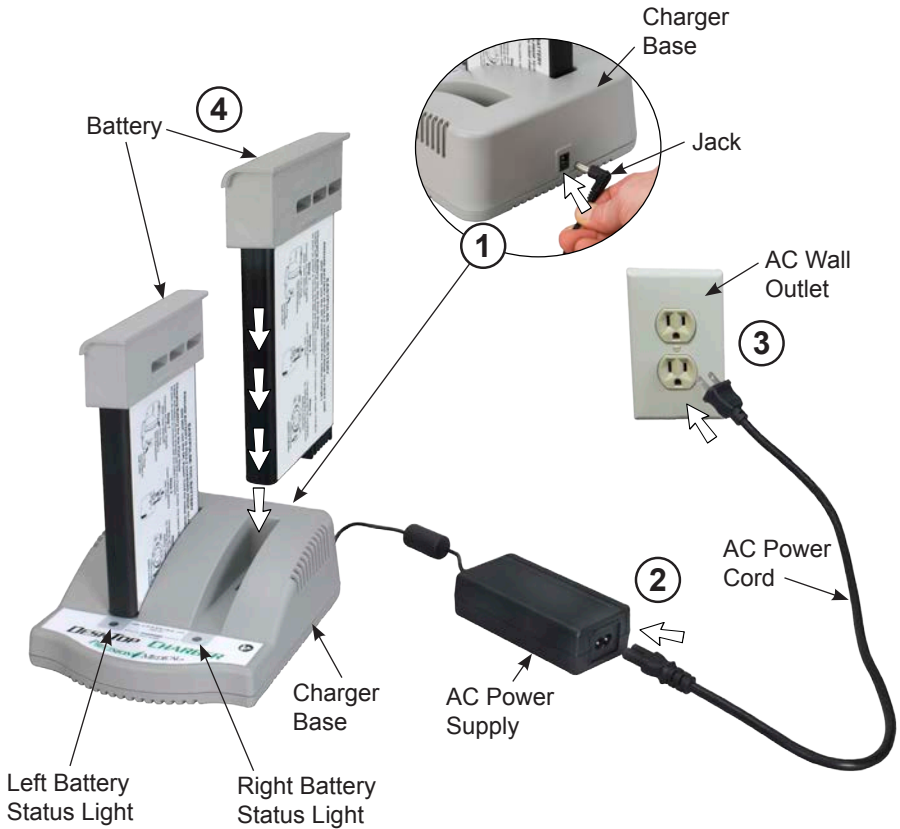

## Instructions

1. Plug AC Power Supply Jack into Charger Base.
2. Connect AC Power Cord into AC Power Supply.
3. Plug AC Power Cord into AC Wall Outlet.
4. Insert Battery into Charger Base.
  - Status Light will turn "On" indicating that the Battery is being charged.
  - Status Light "Blinks" when Battery is fully charged.

## Charge Time

- Charging a Single fully depleted Battery will take approximately 2 hrs to fully charge.
- Charging Two fully depleted Batteries will take approximately 4 hrs to fully charge.

## CAUTION

- Do not open the Desktop Charger or power supply case. There are no serviceable parts inside.
- Do not expose the Desktop Charger or power supply to water or conductive liquids.
- Replace a Battery if it fails to charge or if you notice a decrease in capacity.
- Inspect the AC Power Cord for signs of damage. Discontinue use and replace if damaged.
- Use only the power supply provided with the Desktop Charger.

## REPLACEMENT PARTS

Part Number	Description
506629	AC Power Supply
506961	AC Power Cord
508367	Charger Base
508016	Battery

## SPECIFICATIONS

### Charger Base

<b>Height:</b>	3.00 in (7.62 cm)
<b>Width:</b>	6.20 in (15.75 cm)
<b>Depth:</b>	7.80 in (19.81 cm)

**Weight:** 0.80 lbs (0.36 kg)

**Input:** 18 VDC --- 3.33A

### AC Power Supply

**Weight:** 0.77 lbs (0.35 kg)

**Input:** 100-240V ~ 50-60Hz 1.6-.07A

**Output:** 18 VDC --- 3.33A

Dispose of the device in accordance with local regulations.

For technical specifications call Precision Medical at 1-800-272-7285  
or visit [www.precisionmedical.com](http://www.precisionmedical.com)

## LIMITED WARRANTY AND LIMITATION OF LIABILITY

Precision Medical, Inc. warrants that the Desktop Charger, (the Product), will be free of defects in workmanship and/or material for the following period:

*Desktop Charger - - - - - one (1) year from date of shipment*

*Precision Medical, Inc. is NOT responsible for normal wear and tear, or any neglect or abuse of the product.*

*The customer is responsible for the shipping costs of repairs to Precision Medical, Inc.*

*Precision Medical, Inc. will have in its sole and absolute discretion, the final determination if your product is covered under this limited warranty.*

Should any failure to conform to this warranty appear within the applicable period, Precision Medical, Inc. shall, upon written notification thereof and substantiation that the goods have been stored, installed, maintained and operated in accordance with Precision Medical, Inc.'s instructions and standard industry practice, and that no modifications, substitutions, or alterations have been made to the goods, correct such defect by suitable repair or replacement at its own expense.

ORAL STATEMENTS DO NOT CONSTITUTE WARRANTIES.

The representatives of Precision Medical, Inc. or any retailers are not authorized to make oral warranties about the merchandise described in this contract, and any such statements shall not be relied upon and are not part of the contract for sale. Thus, this writing is a final, complete and exclusive statement of the terms of that contract.

THIS WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ANY WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR OTHER WARRANTY OF QUALITY, WHETHER EXPRESSED OR IMPLIED.

Precision Medical, Inc. shall not under any circumstances be liable for special, incidental or consequential damages including but not limited to lost profits, lost sales, or injury to person or property. Correction of non-conformities as provided above shall constitute fulfillment of all liabilities of Precision Medical, Inc. whether based on contract, negligence, strict tort or otherwise. Precision Medical, Inc. reserves the right to discontinue manufacture of any product or change product materials, designs, or specifications without notice.

Precision Medical, Inc. reserves the right to correct clerical or typographical errors without penalty.