


USER MANUAL

DIGITAL VAC CHECK

Model No: PM760

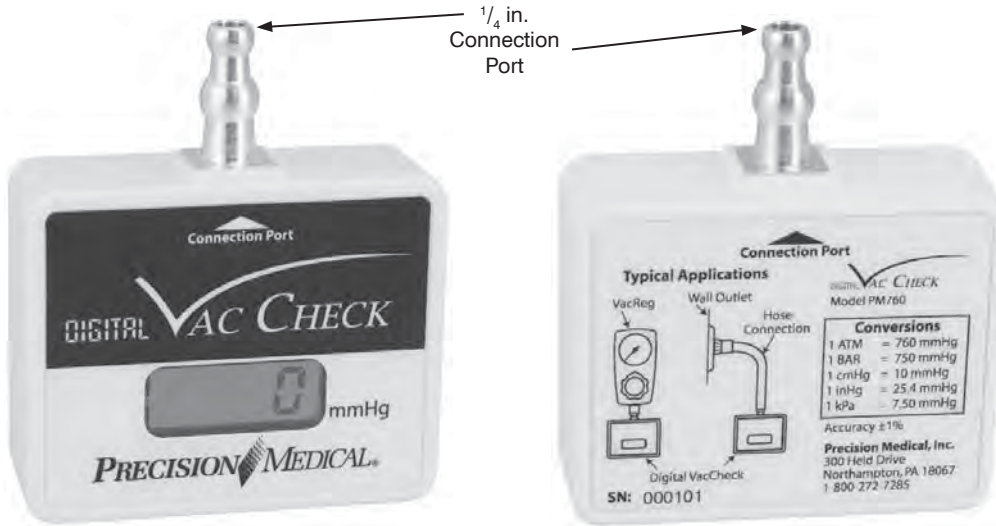

RECEIVING / INSPECTION

Remove the Precision Medical, Inc. Digital Vac Check from the packaging and inspect for damage. If damaged, DO NOT USE. Contact your Provider.

READ ALL INSTRUCTIONS BEFORE USING

This manual instructs the User how to install and operate the Digital Vac Check. This instruction is provided for your safety and to prevent damage to the Digital Vac Check. If you do not understand this manual, DO NOT USE the Digital Vac Check. Contact your Provider.

⚠ WARNING

DO NOT pressurize the Digital Vac Check. This will result in damage to the Vac Check and will void the warranty.

SPECIFICATIONS

Dimensions:

Length (w/ Connection): 3 in (7.6 cm) **Length (w/out Connection):** 2.1 in (5.4 cm)

Width: 2.6 in (6.7 cm) **Weight:** 2.3 oz (65 grams)

Height: 1 in (2.5 cm) **Shipping Weight:** 2.9 oz (83 grams)

Accuracy: +/- 1% of full scale range of 0-760 mmHg

Lithium Ion Battery:

Operating life is 8 to 10 years. Dimming of the display indicates a weak battery. This battery cannot be replaced in the field. Return to Precision Medical, Inc. for battery replacement at a nominal fee plus shipping.

OPERATING INSTRUCTIONS

This Digital Vac Check measures vacuum levels in mmHg. No user interface is required. The Vac Check automatically displays the vacuum level when connected to a source and displays a "0" when disconnected. See back of the Vac Check for Typical Applications.

CLEANING INSTRUCTIONS

1. Disconnect from vacuum source before cleaning.
2. Clean exterior surfaces of the Digital Vac Check with a cloth dampened with a mild detergent and water.
3. Wipe dry with a clean cloth.
4. Store the Vac Check in a clean area free from grease, oil and other sources of contamination.

CALIBRATION

This certifies the PM760 Digital Vac Check, Serial Number _____ has been calibrated according to international measurement standards. If you experience any issues with accuracy, please contact Precision Medical, Inc.

Conversions are listed on the back of Digital Vac Check. See label on the back of Digital Vac Check or reference Conversion Factors below.

CONVERSION FACTORS

1 Atmosphere	=	760 mmHg
1 BAR	=	750 mmHg
1 cmHg	=	10 mmHg
1 inHg	=	25.4 mmHg
1 kPa	=	7.5 mmHg

OR

(Ans.) ATM	=	(V.C.R.) / 760
(Ans.) BAR	=	(V.C.R.) / 750
(Ans.) cmHg	=	(V.C.R.) / 10
(Ans.) inHg	=	(V.C.R.) / 25.4
(Ans.) kPa	=	(V.C.R.) / 7.5

V.C.R. = Vac Check Reading
Ans. = Answer or Desired Value

RETURNS

Returned products require a Returned Goods Authorization (RGA) number, contact Precision Medical, Inc. All returns must be packaged in sealed containers to prevent damage. Precision Medical, Inc. will not be responsible for goods damaged in transit. Refer to Precision Medical, Inc. Return Policy available on the Internet, www.precisionmedical.com.

LIMITED WARRANTY AND LIMITATION OF LIABILITY

Precision Medical, Inc. warrants that the Medical Digital Vac Check (the Product) will be free of defects in workmanship and/or material for the following period:

Digital Vac Check One (1) year from date of shipment.

Battery One (1) year from date of shipment.

Should any failure to conform to this warranty appear within the applicable period, Precision Medical, Inc. shall, upon written notification thereof and substantiation that the goods have been stored, installed, maintained and operated in accordance with Precision Medical, Inc.'s instructions and standard industry practice, and that no modifications, substitutions, or alterations have been made to the goods, correct such defect by suitable repair or replacement at its own expense.

ORAL STATEMENTS DO NOT CONSTITUTE WARRANTIES.

The representatives of Precision Medical, Inc. or any retailers are not authorized to make oral warranties about the merchandise described in this contract, and any such statements shall not be relied upon and are not part of the contract for sale. Thus, this writing is a final, complete and exclusive statement of the terms of that contract.

THIS WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ANY WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR OTHER WARRANTY OF QUALITY, WHETHER EXPRESS OR IMPLIED.

Precision Medical, Inc. shall not under any circumstances be liable for special, incidental or consequential damages including but not limited to lost profits, lost sales, or injury to person or property. Correction of non-conformities as provided above shall constitute fulfillment of all liabilities of Precision Medical, Inc. whether based on contract, negligence, strict tort or otherwise. Precision Medical, Inc. reserves the right to discontinue manufacture of any product or change product materials, designs, or specifications without notice.

Precision Medical, Inc. reserves the right to correct clerical or typographical errors without penalty.