

# live ACTIVE Five

## Portable Oxygen Concentrator

### Live Active Five® Starter Kit includes:



3-year warranty on concentrator

1-year warranty on sieve bed and battery

The Live Active Five Power Bundle includes the Live Active Five Starter Kit items plus an additional Lithium-ion Battery.

Battery charging kit, additional batteries and extended warranty are also available for purchase.



LiveActive5.com

Talk to your provider about insurance coverage or purchase options for your Live Active Five!



300 Held Drive • Northampton, PA 18067  
800-272-7285 • precisionmedical.com



508646 Rev 2 Printed in USA

DON'T JUST LIVE. LIVE ACTIVE.

# live ACTIVE Five

## Portable Oxygen Concentrator



LiveActive5.com





# DON'T JUST Live Live ACTIVE



Weights just 5 lbs



FAA approved for air travel



Great for cruise ships

Makes traveling with oxygen easier

No more dragging tanks or change overs

Lightweight, compact design

One touch start

5 pulse flow settings

6 hours of battery duration

Versatile carry bag for hands-free wear

## Battery Duration at any breath rate

Setting 1	6.5 hours
Setting 2	4.3 hours
Setting 3	2.7 hours
Setting 4	2.0 hours
Setting 5	1.5 hours

## Ease, comfort and peace of mind

An easy-to-understand display screen makes the device simple to use. One push of a button turns on and adjusts the pulse flow setting. An ergonomic, dual-curve design allows for comfortable wear on either side of the body.

## Highest oxygen purity possible

With Precision Medical's proprietary Vacuum Pressure Swing Adsorption (VPSA) technology, you'll receive the highest oxygen purity possible (95.5%). The VPSA technology optimizes device performance, producing therapeutic oxygen for a longer time and reducing the frequency of sieve bed replacements.

## Easy-to-change battery

Change batteries quickly and easily while on the go with the convenient, top-access battery. With more than 6 hours of duration with each battery on setting 1, the single battery design allows you to enjoy activities longer without the added weight of bigger or bulkier batteries.

## Multiple charging options

Live Active Five can be charged by plugging into any electrical outlet or the DC outlet in a car, RV or boat. Quickly charge additional batteries using the optional Live Active Five Battery Charger.

## Low maintenance

Designed for minimal maintenance, the device alerts you when it is time to replace the sieve beds. The all-in-one sieve bed replacement can be changed by you or your durable medical equipment provider.

